



CYTOLOID™ MUCOLYTIC REAGENT PROCEDURE

Item# CYCLO

(Revised 04/23/11)

SPECIMEN: Fresh sputum

PROCEDURE:

1. Transfer specimen into a 50 ml conical centrifuge tube.
2. Add 2 volumes of **CYTOLOID™** for each volume of specimen and combine thoroughly using a vortex mixer.
3. Stop mixing and incubate the mixture at room temperature for 30 to 60 minutes with periodic vortex mixing.
4. Centrifuge the homogenized mixture for 5 minutes at 500 g.
5. Prepare smears using the cell button.
6. Completely air dry smears.
7. Fix smears through two changes of 95% Reagent Alcohol.
8. Stain smears as usual.

REFERENCE: ACTA CYTOLOGICA, The Journal of Clinical Cytology and Cytopathology; Vol. 39; 1995; 1137-1140.

Copyright ©2011 American MasterTech Scientific Incorporated. All rights reserved.

No part of these pages may be used for any purpose other than personal use. Therefore, reproduction, modification, storage in a retrieval system or retransmission, in any form or by any means, for reasons other than personal use, is strictly prohibited without prior written permission.